UK: HEARING AID MAINTENANCE

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**UK: PHONICARE Moist CLEANING Wipes with Lemon fragrance:
Cleaning wipe dispenser 90/ 50 pcs**

1. Open the dispenser and take one wipe.
 2. Clean the hearing aid and/ or earmold with the wipe.
 3. Dry the hearing aid and/ or earmold with a soft, dry cloth.
 4. The cleaning wipe is for single use. Throw it away after use.
- Hearing aids and earmolds should be cleaned at least twice a week.
 - **Attention!** Flammable liquid and vapor | Avoid eye contact | Keep out of reach of children | For external use only.

100g disinfection liquid contain:

0,25g Didecyldimethylammoniumchloride/ (DDAC), 0,25g Alkyl (C12-16) dimethylbenzylammoniumchloride/ Alkyl (C12-(14) dimethyl(ethylbenzyl) ammoniumchloride/ Lemon concentrate.

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0,25g Didecyldimethylammoniumchlorid/ (DDAC), 0,25g Alkyl (C12-16) dimethylbenzylammoniumchlorid/ Alkyl (C12-(14) dimethyl(ethylbenzyl) ammoniumchlorid/ Zitronenkonzentrat.

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(DDAC), 0,25g Alkyl (C12-16) dimethylbenzylammoniumchloride/ Alky I(C12-(14) dimethyl(ethylbenzyl) ammoniumchloride/ Lemon concentrate.

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Refill

GB: PHONICARE moist CLEANING wipes with lemon fragrance Individually packed 25 pcs

1. Open the packing and take out the wipe.
 2. Clean the hearing aid and/ or earmold with the wipe.
 3. Dry the hearing aid and/ or earmold with a soft, dry cloth.
 4. The cleaning wipe is for single use. Throw it away after use.
- Hearing aids and earmolds should be cleaned at least twice a week.
 - **Attention!** Flammable liquid and vapor | Avoid eye contact | Keep out of reach of children | For external use only.

100g disinfection liquid contain:

0,25g Didecyldimethylammoniumchloride/ (DDAC), 0,25g Alkyl (C12-16) dimethylbenzylammoniumchloride/
Alkyl (C12-(14) dimethyl(ethylbenzyl) ammoniumchloride/ Lemon concentrate.
Tested to European Norms (EN Standards). More information upon request.

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UK: Phonicare Cleaning Spray with lemon fragrance:

Spray bottle 30ml

1. Spray the product on a clean, soft, and dry cloth.
**Never spray the product directly on the hearing aid!*
 2. Rub the cloth over the surface of the hearing aid and clean it.
 3. After cleaning the hearing aid, dry it with a soft, dry cloth.
- Hearing aids and earmolds should be cleaned at least twice a week.
 - **Attention!** Flammable liquid and vapor | Avoid eye contact | Keep out of reach of children | For external use only.

100g disinfection liquid contain:

0,25g Didecyldimethylammoniumchloride/ (DDAC), 0,25g Alkyl (C12-16) dimethylbenzylammoniumchloride/ Alkyl (C12-(14) dimethyl(ethylbenzyl) ammoniumchloride/ Lemon concentrate.
Tested to European Norms (EN Standards). More information upon request.

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0,25g Didecyl dimethyl ammonium chloride/ (DDAC), 0,25g Alkyl (C12-16) dimethyl benzyl ammonium chloride/ Alkyl (C12-(14) dimethyl (ethyl benzyl) ammonium chloride/ Lemon concentrate.

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UK: Phonicare Cleaning tablets with active oxygen:

Cleaning tablets 30 pcs

1. Separate the earmold from the hearing aid.
 2. Place the cleaning tablet in the cup and add warm water until it is 2/3 full.
 3. Place the earmold in the cup's plastic basket and put it in the cup.
(never put the hearing aid in the liquid!)
 4. Let the earmold clean for minimum 30 minutes. (Overnight possible)
 5. After cleaning, rinse the earmold with warm water and dry it thoroughly before reconnecting to the hearing aid.
 6. Use the Phonicare air bulb to dry the earmold/ tubed
 7. Cleaning tablets are for single use.
- Earmolds/ tubes should be cleaned at least once a week.
 - **Attention!** Avoid eye contact with the liquid | Keep out of reach of children | For external use only | Never put your hearing aid in the liquid!

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